

METASARAL PAD



ABOUT THE PRODUCT

The Metatarsal Pad is designed to relieve pressure and pain in the ball of the foot. It provides gentle cushioning under the metatarsal area, helping to reduce discomfort from conditions like metatarsalgia, bunions, or long hours of standing and walking. Soft, flexible, and easy to use, it can be placed inside most shoes for instant support and comfort.

BENEFITS

- The simple loop fits around the toe to hold this gel pad in place under the forefoot, relieving pressure on the plantar forefoot and metatarsal bones.
- Supports metatarsal-forefoot area while body weight is transferred from heel-via-arch to forefoot (gait cycle).
- Useful for post-forefoot surgery care, relieving metatarsalgia, and managing progressive pain towards the ankle.

HOW TO USE

- Place the pad under the ball of your foot, just behind the toes.
- Adjust until it feels comfortable and supports the forefoot.
- Use inside shoes or slippers for daily cushioning and relief.

Instructions : If the product gets dirty or sticky, clean with talcum powder, rinse with mild shampoo and water, shade dry, and avoid direct heat.

Safety : Keep away from heat and sharp objects.

THANK YOU FOR YOUR PURCHASE — YOUR COMFORT MATTERS TO US.

**FEEL FREE TO CONTACT US FOR MORE INFORMATION
WWW.LIVISH.IN
PH NO- +91 99620 94754**