

# TOE SEPRATOR WITH 1 LOOP & METASARAL PAD



## ABOUT THE PRODUCT

A gel toe spreader with a loop and metatarsal pad is a specialized device that addresses multiple foot issues at once. The gel toe spreader portion, secured by a loop around a toe (usually the big toe), helps to gently separate toes and prevent friction. The attached metatarsal pad cushions the ball of the foot, providing support and relieving pressure on the metatarsal bones, which is especially helpful for conditions like metatarsalgia or a painful forefoot

## BENEFITS

- Function as cushion to absorbs excess pressure over the metatarsal area and provides an area of very soft padding for the balls of the feet. Fits width of the forefoot.
- Straighten and align toes by gentle constant pressure while spreading toes and reduce friction between toes preventing infections.
- Relieve pain of overlapping toes/toe drift and painful bunions.

## HOW TO USE

Slide the loop over your toe, position the separator between the toes, and place the metatarsal pad under the forefoot for cushioning.

**Instructions :** If the product gets dirty or sticky, clean with talcum powder, rinse with mild shampoo and water, shade dry, and avoid direct heat.

**Safety :** Keep away from heat and sharp objects.

**THANK YOU FOR YOUR PURCHASE — YOUR COMFORT MATTERS TO US.**

**FEEL FREE TO CONTACT US FOR MORE INFORMATION**

**WWW.LIVISH.IN**

**PH NO- +91 99620 94754**