


WHEEL CHAIR SUPPORT (SMALL SQR)

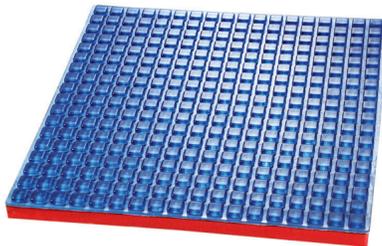

ABOUT THE PRODUCT

A wheelchair support pad featuring small square tiles is a therapeutic cushion designed to prevent pressure ulcers and provide stability for wheelchair users. It is constructed from a grid of small, individual gel or foam square-shaped cells. This mosaic design allows the cushion to precisely conform to the user's anatomy, enabling superior weight distribution and minimizing pressure concentration on bony areas like the tailbone and hips. The spaces between the square tiles facilitate airflow, helping to keep the skin cool and dry, which is critical for long-term comfort and skin health.

BENEFITS

- Small square tile gel layer 10 mm thick & 1 inch soft PU foam with cloth cover base provides ultimate comfort and better posture.
- Reduces Skin temperature, heat flux and relative humidity.
- Temperature remains static under the ischial tuberosities and posterior thighs.

HOW TO USE

- Place the support cushion on the wheelchair seat with the gel side facing up.
- Adjust it so it sits centrally under the hips for even pressure distribution.
- Sit comfortably, ensuring the cushion covers the pressure points for maximum support and comfort.

Instructions : If the product gets dirty or sticky, clean with talcum powder, rinse with mild shampoo and water, shade dry, and avoid direct heat.

Safety : Keep away from heat and sharp objects.

THANK YOU FOR YOUR PURCHASE — YOUR COMFORT MATTERS TO US.

**FEEL FREE TO CONTACT US FOR MORE INFORMATION
WWW.LIVISH.IN**

PH NO- +91 99620 94754