

# WHEEL CHAIR SUPPORT (BIG SQR)



## ABOUT THE PRODUCT

A wheelchair support pad featuring big square tiles is a specialized cushion designed for pressure distribution and stability in wheelchair seating. It is typically constructed from a series of larger, individual gel or foam square cells that work together to conform to the user's body. This design provides excellent support by minimizing concentrated pressure on bony areas and maximizing the surface area over which weight is spread. The larger squares offer a balance between conformance and stability, while the spaces between them assist in airflow and moisture control, making it a durable and comfortable option for extended wheelchair use

## BENEFITS

- Big square tile gel layer 22 mm thick & 1 inch soft PU foam with cloth cover base provides ultimate comfort and better posture.
- Reduces Skin temperature, heat flux and relative humidity.
- Temperature remains static under the ischial tuberosities and posterior thighs.

## HOW TO USE

- Place the big square support cushion on the wheelchair seat with the gel side facing up.
- Position it centrally on the seat, ensuring it fully covers the sitting area.
- Sit down slowly and adjust your position so the cushion supports the hips and reduces pressure on sensitive areas.

**Instructions :** If the product gets dirty or sticky, clean with talcum powder, rinse with mild shampoo and water, shade dry, and avoid direct heat.

**Safety :** Keep away from heat and sharp objects.

**THANK YOU FOR YOUR PURCHASE — YOUR COMFORT MATTERS TO US.**

**FEEL FREE TO CONTACT US FOR MORE INFORMATION**  
**WWW.LIVISH.IN**  
**PH NO- +91 99620 94754**