

GEL MATTRESS



ABOUT THE PRODUCT

Gel Mattress is designed to provide ultimate comfort and pressure relief. Made with Progel technology, it evenly distributes body weight, reducing stress on the back, hips, and joints. The gel layer keeps you cool, cushioned, and supported throughout the night. Ideal for patients, seniors, or anyone needing extra comfort, it is durable, skin-friendly, and easy to clean — ensuring restful, pain-free sleep every day.

BENEFITS

- Provide comfortable position for Comatose patient.
- Foldable three slabs of 18 mm thick corrugated gel layer pasted on 2" soft PU foam base with cloth cover provides ultimate comfort and better posture.
- Reduces Skin temperature, heat flux and relative humidity.
- Effectively reduce ischemic tissue damage & incidence of pressure sores. Last long and infection free.

HOW TO USE

- Spread the gel mattress evenly on top of your regular mattress or hospital bed surface.
- Gently help the patient lie down, ensuring their body rests fully on the gel area for even pressure distribution.
- Make sure it lies flat with no folds. Reposition the patient periodically for maximum comfort and pressure-sore prevention.

Instructions : If the product gets dirty or sticky, clean with talcum powder, rinse with mild shampoo and water, shade dry, and avoid direct heat.

Safety : Keep away from heat and sharp objects.

THANK YOU FOR YOUR PURCHASE — YOUR COMFORT MATTERS TO US.

FEEL FREE TO CONTACT US FOR MORE INFORMATION
WWW.LIVISH.IN
PH NO- +91 99620 94754